

#4539 Function Keys in Module Aligner 3D

Since Release 3.2.90 (209) in Modul Aligner 3D the following extended function keys can be used:

- F8: Go to next movement limit maxilla/mandible
- SHIFT+F8: Go to next movement limit maxilla
- CTRL+F8: Go to next movement limit mandible
- F9: New step
- F10: Compensate collisions

If tooth movement limitations are defined, the treatment steps between initial and goal situation can be found by multiple function button sequence [SHIFT|CTRL+]F8+F9+F10 (in case, the manual correction per step can be limited to the compensation of collisions between crowns).

For F10 please note that the crown collisions are only compensated inbetween adjacent crowns multiselected in the object list.

From:
<https://www.onyxwiki.net/> - [OnyxCeph^{3™} Wiki]



Permanent link:
<https://www.onyxwiki.net/doku.php?id=en:4539>

Last update: **2017/10/22 11:41**