

Module Add Image 3D




Program version	BASIC	2D PRO	3D PRO	LAB	OMS
Availability					

In Module Add Image 3D different types of 3D data sets can be imported from pre-defined image sources, classified acc. to [session](#) and [image type](#) and completed with [attributes and comments](#) before saved to the Onyx database.

First Steps

1. Activate "Clipping (Line)" in tab "Clip Object"
2. Place Landmarks at the model border, last point close to the first one
3. Press button "Trim" and confirm the correct trimming
4. Select and add socket in tab "Model Base"
5. Save as new finding

Module user interface

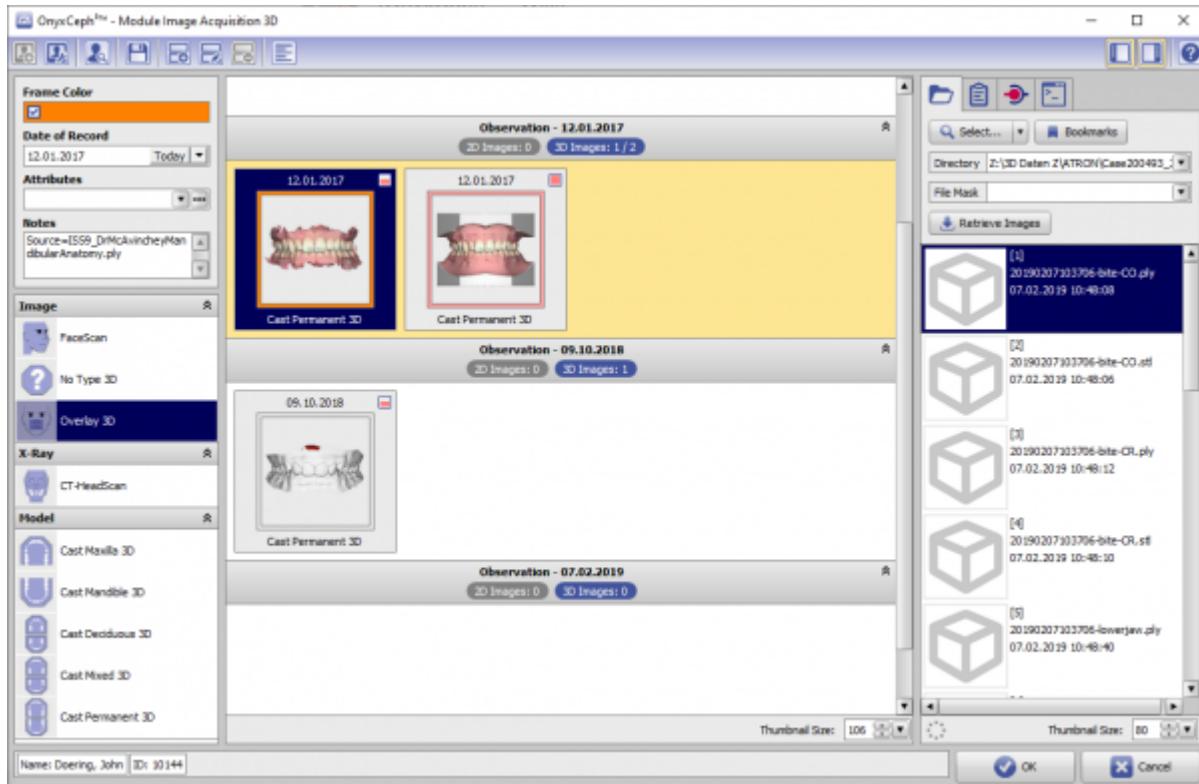

Links

	Module Handout Image Import 3D
	Import Adjust Scan
	https://youtu.be/TAAGkmFIKD4
	https://youtu.be/Q9R9V5QgEoE

From:
[http://www.onyxwiki.net/ - \[OnyxCeph³™ Wiki\]](http://www.onyxwiki.net/)



Permanent link:
http://www.onyxwiki.net/doku.php?id=en:add_image_3d&rev=1613246940

Last update: **2021/02/13 21:09**