

Module Bonding Jigs 3D



Program version	BASIC	2D PRO	3D PRO	LAB	OMS
Availability	×	×			



not included



included by

default



optionally


included

In module Bonding Jigs 3D module, occlusal transfer jigs can be constructed to transference the virtually planned bracket positions using metallic standard bracket adapters.

First Steps

1. Open Module with bracket-attached Model
2. Select Crowns
3. Assign adapter and Jig parameters to crowns
4. Construct Jigs individually or connected
5. Arrange Jigs for manufacturing / export
6. Save planning as project and/or as finding

Module user interface

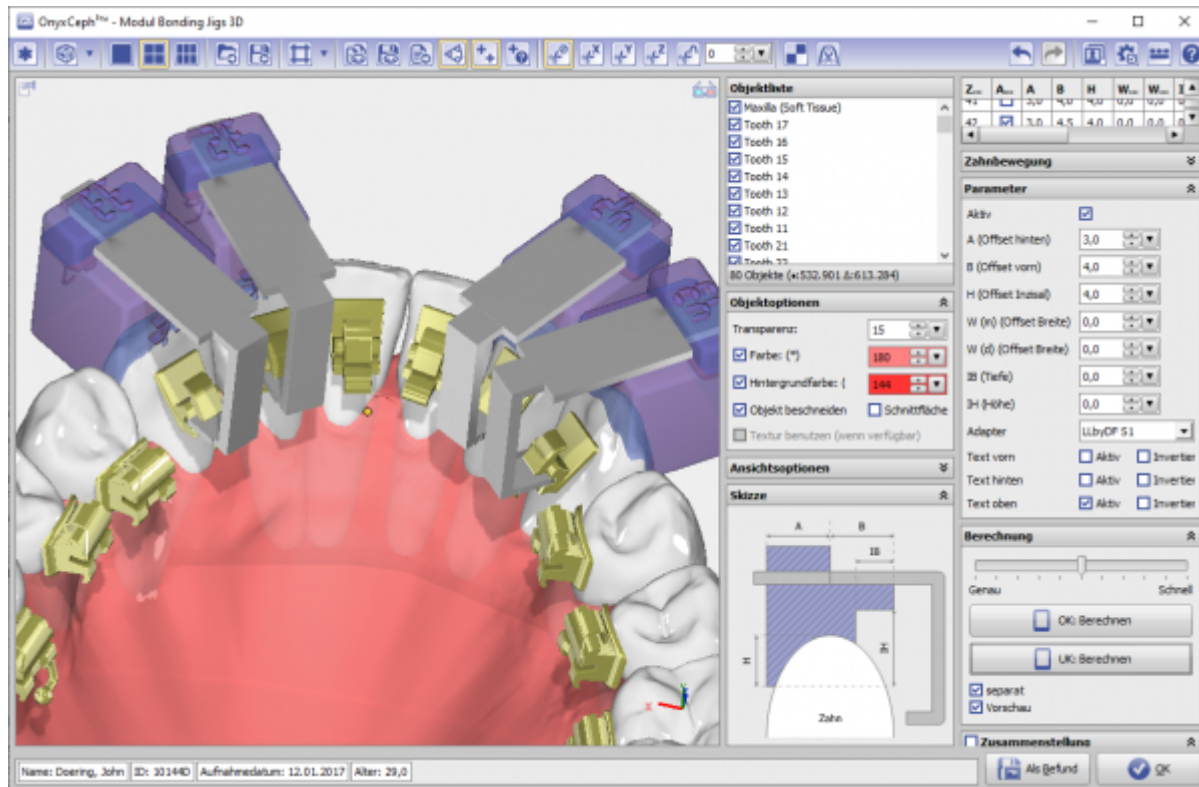

Note



Special metal adapters are required for the use of the module. These adapters are currently no longer offered by the corresponding manufacturers.

From:
<http://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:
http://onyxwiki.net/doku.php?id=en:bonding_jigs_3d

Last update: **2023/02/20 17:05**

