

Module Combine 2D



Program version	BASIC	2D PRO	3D PRO	LAB	OMS
Availability					


 not included
 included by
 default
 optionally
 included

Module Combine 2D can combine two images with different but overlapping content. The second image behind the first image is shown completely or in a self-defined region of interest using common reference points with selectable transparency. In addition, the tracing of one of the two images can be overlaid.

First Steps

1. Open module with 2 images
2. Select or digitize correspondin landmarks
3. Define region of interest
4. Switch to tab |Combine|
5. Optionally, modify superimposition
6. Save result as project and/or as finding

Module user interface

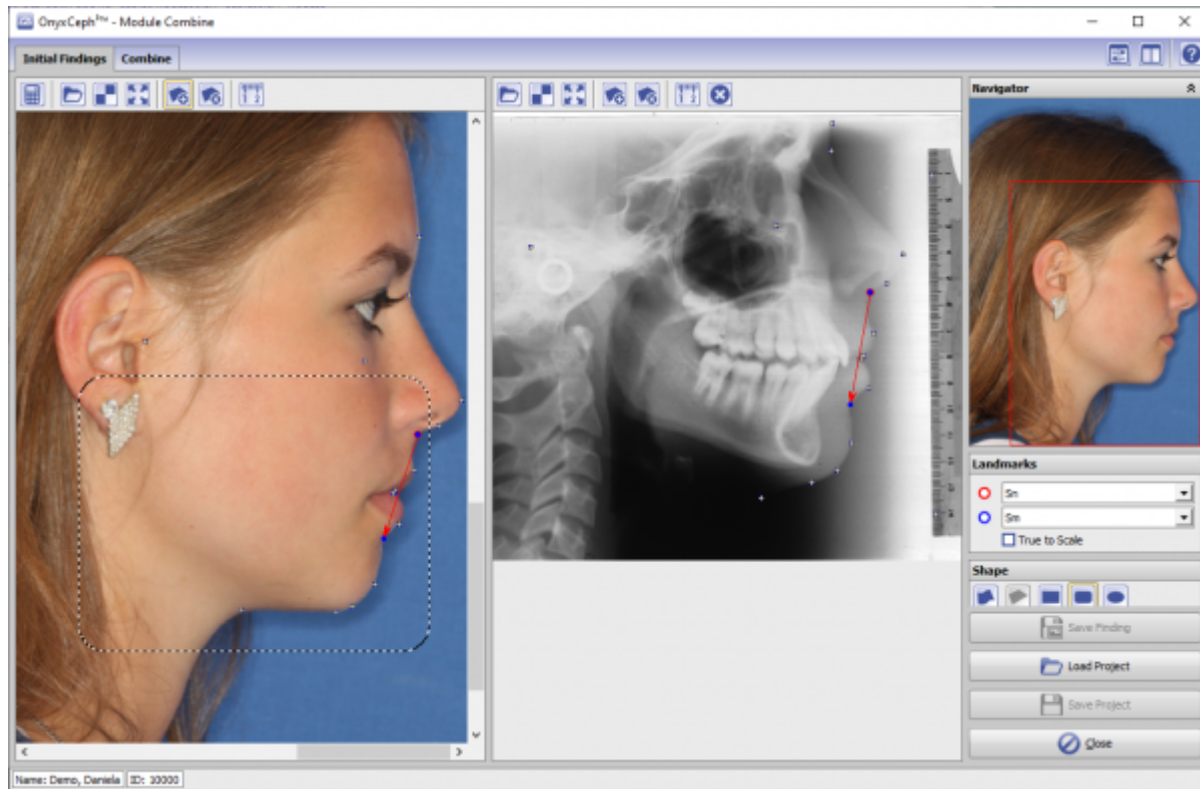

Links



n/a

From:

<http://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:

http://onyxwiki.net/doku.php?id=en:combine_2dLast update: **2023/03/06 12:50**