Module FA_Bonding

First Steps

- 1. Place bracket point from list onto the correct tooth, model rotation determines up vector
- 2. Select bracket for point or load bracket set
- 3. Adjust bracket using the 3D navigators or the line-up tool
- 4. Harmonize crown distance
- 5. Save as project and new finding

Module user interface



Links

لم	https://image-instruments.de/Onyx3/Handout_IDB_FA_BT_DE.pd
►	https://youtu.be/bgCUIIm2YKw
Þ	https://youtu.be/uulEFoQ_IV0

From:

http://www.onyxwiki.net/ - [OnyxCeph³™ Wiki]

Permanent link: http://www.onyxwiki.net/doku.php?id=en:fa_bonding&rev=1584709182

Last update: 2020/03/20 13:59

