

System Options - Tab |Email|

On system options tab |Email|, account settings for the direct communication with an Email server and for sending email attachments can be configured.

System Options

Patient Data

Image Types

Sessions

Galleries

Printouts

Forms

Display

Interfaces

Third Party Tools

Event

Upload Data

Email

☒ Direct Dispatch

Server (SMTP)

Connection Security

TCP/IP Port

User Name

Password

Sender - Name

Sender (FROM)

Copy (CC)

Blind Copy (BCC)

Test

None

25

Standard: 25

.....

... X

... X

Send E-Mail...

Export - Attachment

Directory

☒ Fix Filename

☒ Password

☐ Maximum Image Size

☒ Execute

D:\ZI_Prog\OnyxEmails

1024 x 768

Send E-Mail

?

Help

Close

[Click on tab to switch to related topic]

Panel Direct Dispatch

At various situations it is possible to send email messages - both with and without attachments.
Here you can store and test the access data for the email account to be used for the [sending type](#) [direct mail] to be used.

Panel Export - Attachment

This specifies settings for sending email attachments:

Entry	Meaning
Directory	Directory for caching file attachments
Fixed file name	Definition of a fixed file attachment name
Password	Password default (can be modified before sending)
Maximum image size	Maximum single image size for file attachments
Execute	Event when filling export directory

From:
<https://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:
https://onyxwiki.net/doku.php?id=en:tab_email

Last update: **2022/06/24 08:51**

