

Tracing Types 2D

OnyxCeph³™ supports the following tracing types 2D:

Type
Photo Frontal
Photo Lateral
Photo Smiling
Ceph Frontal
Ceph Lateral
Hand Xray
Cast Permanent Dentition 2D
Cast Mixed Dentition 2D
Cast Deciduous Dentition 2D
Panoramic View
SMV
Not Defined 2D

Each 2D image type has exactly assigned one tracing type 2D. Thus, in module [Digitize](#), all analyses pre-defined for this tracing type are offered for selection and application. The tracing type assignment can be modified via menu [Options|System Options](#) on tab |Image Types 2D|.

From:

<https://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:

https://onyxwiki.net/doku.php?id=en:tracingtypes_2d

Last update: **2020/07/23 11:08**

