


# Ceph Tracing

### ◀ TP Key Users


## 1. Open Tracing Module from Thumbnail

Click on main menu icon [Digitize] to open tracing module window.

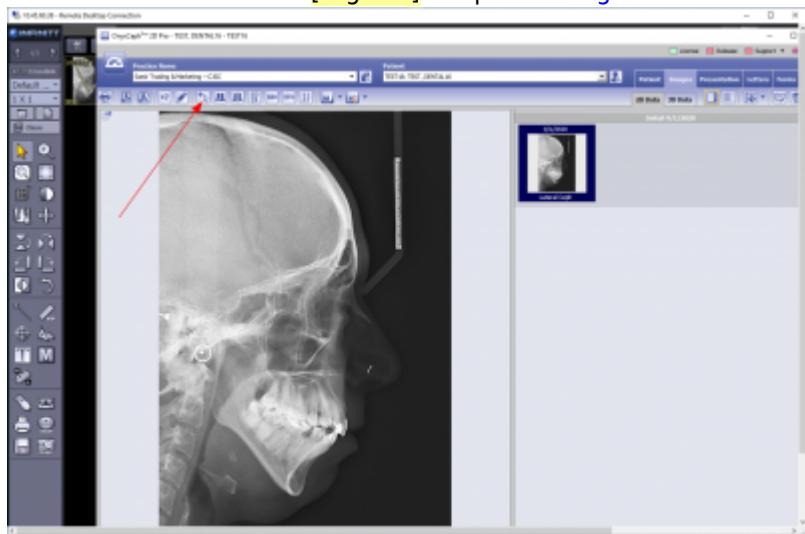


## 2. Select Analysis

Make sure [only] lateral ceph analysis [Riyahd] is selected - otherwise, [select analysis from Analysis List](#)  
Make sure [image calibration](#) is correct.

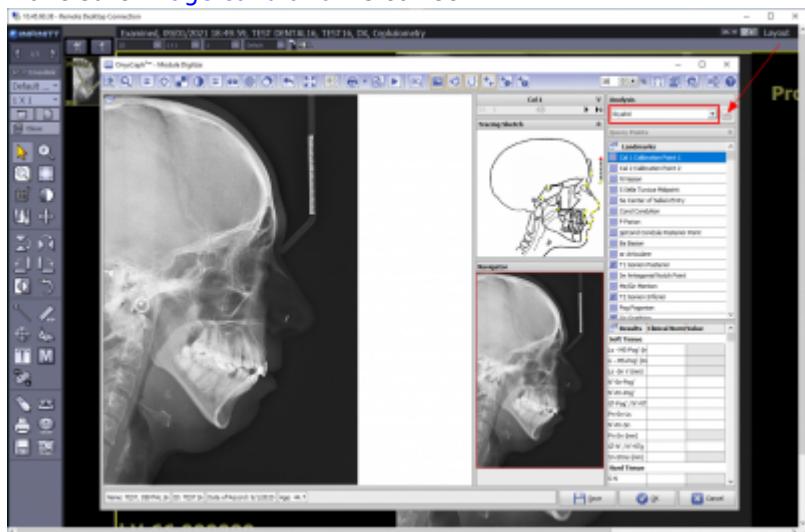


### 3. Digitize Ceph Landmarks

Digitize landmarks in landmark list by right clicking the related position in the image.  
Correct landmark position by left mouse drag&drop.

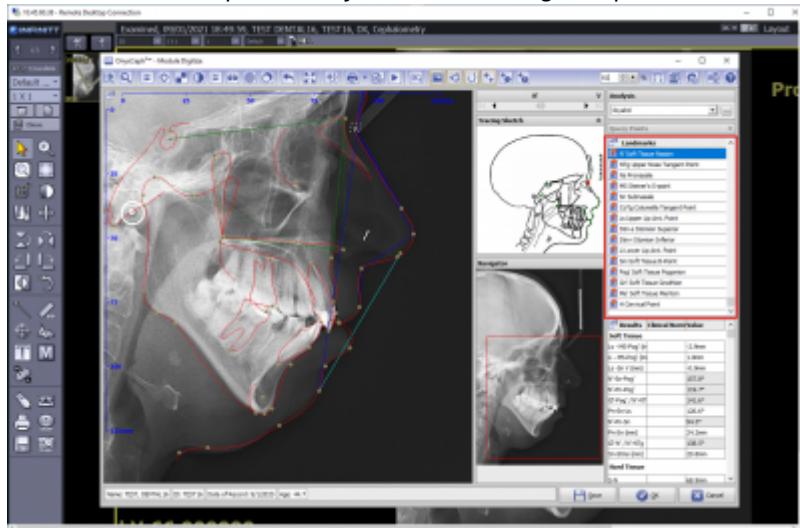


### 4. Adjust Profile Template

Optionally, adjust profile template by Bezier curve.

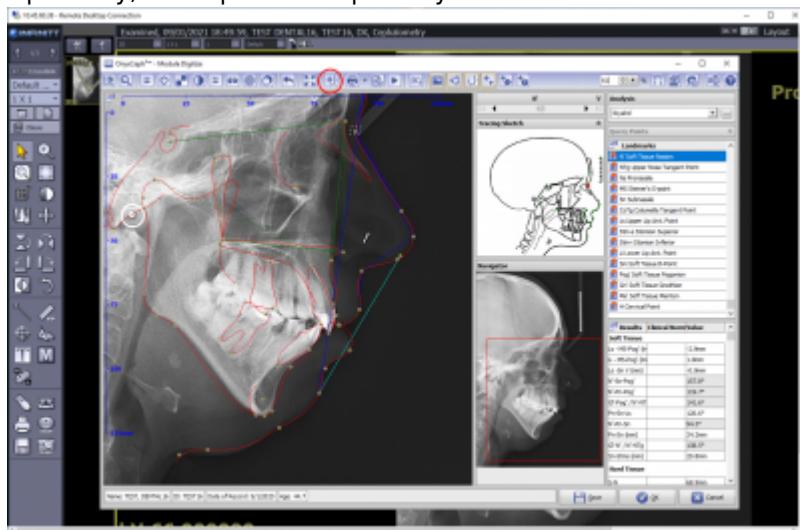

### 5. View Results

View results in [Result List] - open full screen list via context menu.

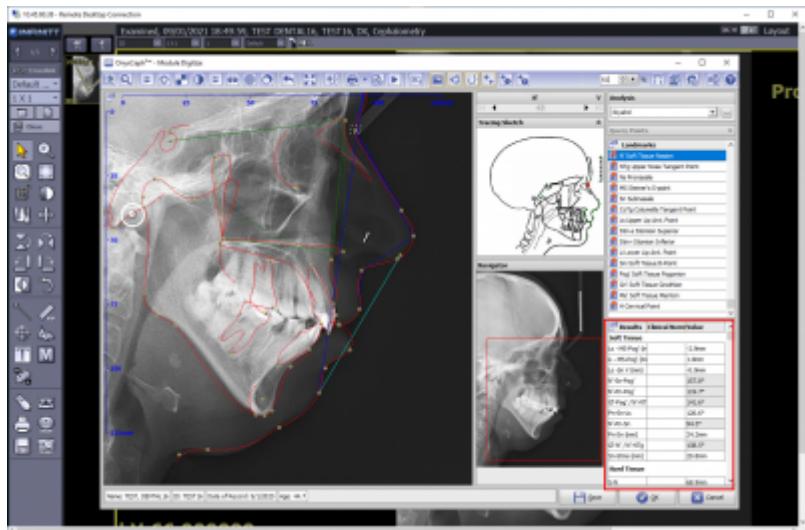

## Links



◀ TP Key Users

From: <http://www.onyxwiki.net/> - [OnyxCeph<sup>3</sup>™ Wiki]

Permanent link:  
[http://www.onyxwiki.net/doku.php?id=moh:tp\\_keyuser3](http://www.onyxwiki.net/doku.php?id=moh:tp_keyuser3)

Last update: **2022/03/15 12:32**

