



This page is not fully translated, yet. Please help completing the translation.
(remove this paragraph once the translation is finished)


Module Image Comparison 2D

In module Findings Comparison 2D, two or more findings of the same image type can be compared numerically and graphically. The numerical comparison is provided as a tabular comparison of the cephalometric measurement values of the selected analysis with or without clinical norms. The graphic comparison can be done as a superimposition of the tracing, as a segmented overlay and as a superimposition of harmony boxes. The tracings can be superimposed at reference points or (by means of gray value correlation) on growth-stable skeletal structures.

First Steps

1. Open module with 2 or more findings of the same image type
2. Configure numerical comparison
3. select type and parameters for graphical superimposition
4. Adjust graphical superimposition

Module user interface

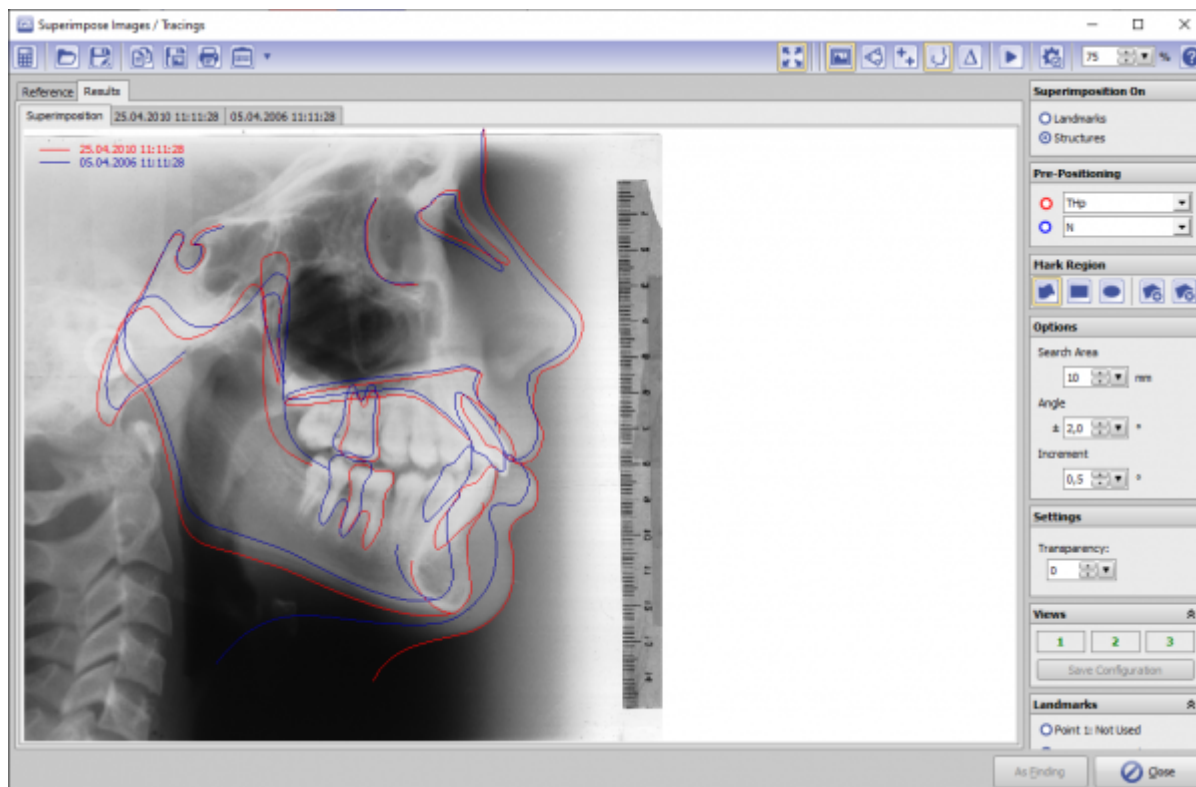

Links



From:

<https://www.onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:

https://www.onyxwiki.net/doku.php?id=en:image_comparison_2d&rev=1584549120

Last update: **2020/03/18 17:32**

