[OnyxCeph³[™] Wiki] - http://onyxwiki.net/

Main Window Tab |Letters|

Presentation Patient Images **Recycle Bin**

The [Letters] tab allows generating, patient-related saving or printing of individual pre-defined letters for patients or colleagues.

User Interface

OnyxCeph ¹ ™ 3D Lab - Demo, Daniela - 10000			- 🗆 X
		Licens	e 📕 Release 📕 Support 🔻 🛞 Language
Practice Name Image Instruments	Patient 10000: Demo, Daniela	Patient Images Presentation	Letters Forms Online Recycle Bin
E E E POF * C Template: Onyx_Faliplanungsreport 3			🖙 🗊 * 🕜
💞 🖻 🍋 @DFGothic-EB 👱 10 🗮 🕨			
	OnyxCeph	THE ^	☑ 11.01.2016 - Cast Permanent 3D
PATIENT	Fall		
Daniela Demo 28,9 Years; Female	Modellnummer: MN 55055 Benutzer: Dr. V Müller		
ID: 10000	Kommentar: Scan-Datum: 20.08.2019		
Praxis/ Klinik	Druckdatum: 20.08.2019		
Malokklusion Front 20.08.2019	V.T.O.3D Front 20.08.2019		
		_	
Therease	THE PARTY OF		
PCC/ Street	200000000000000000000000000000000000000		
	and a local balla		
			Printer:
	lavalla Jakakas names - OK	v	Printer 1
Name: Demo, Daniela ID: 10000 Birthday: 19.09.1990 Age: 28,9 Created: 15.10	0.2008 Changed: 28.09.2018	Workstation: User [R	(K8] Server: ONYXD83-[DAVID-LG] @DAVID-LG

Letter Templates

For generating and managing of letters, whose contents refer to diagnostic information or planning details of the database, pre-defined layout drafts can be selected from the [Forms] pull-down of the |Letters| tab. The linked images are displayed in the panel right of the preview area. The items pre-selected can be replaced by alternative images of the same image type by means of the [Select Image] button. The [Preview] button below the image list opens the letter with the image data selected in the preview panel at the left. If there are editable text fields in the layout, they can be completed or edited in the preview mode by the help of the text options above the preview area.

The letters displayed in the preview window can be directly printed by the [Print] button or saved in the database under the layout draft's and the current date by the [Save Data] button. Already saved (completed) letters can be displayed and edited by the icon button [Open]. In addition the saved letters appear in the document tree on the [Patient] tab and, similar to all other entries, can be directly called there.

There is no context menu on the |Letters| tab work space.

From: http://onyxwiki.net/ - [OnyxCeph³™ Wiki]

Permanent link: http://onyxwiki.net/doku.php?id=en:maintab_letters

Last update: 2023/02/15 13:38

