
Saving and Exchanging Case Planning

Any planning module (e.g., Aligner 3D) is opened from a "*parent*" finding thumbnail that meets the corresponding requirements (e.g. for Aligner planning, the finding has to be a segmented permanent cast).

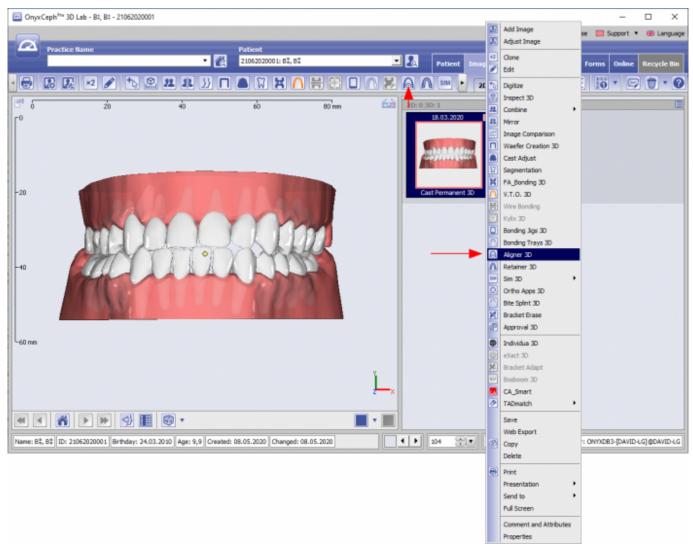

[Example planning module Aligner 3D]

	🔄 🛯 🐨 🔳 📕 👪 🗖 .	₿⋣・₿₿₿	8 🖻 🗄 🕇 🖬	S 🗟 🔩	to ∩ 🔹	4° 4 [×] 4 [×]	2 🖴 🏲 🔳	6 =	
1.8	20	40	60	80 mm	Tooth 21	¥	Attachments		
1		- T			14 4	F - Fi	Tooth :		
	Save entire plannin				Object List	\$	Tooth :	_	
	process as Projec		-		MAAN	ALLANYE	Tooth :	-	
	process as Projec				34 Objects (+: 291	015 0-479 747	Tooth :	-	
							Tooth :		
		and the second			Object Options	¥	🙎 Tooth :		
	Save Project	×			View Options	\$	Tooth :		
20	Available Objects					Flats -	Tooth : RealDen		
2	Tx01	08.05.2020 09:00:31			Overtracing	20 💮 💌	Tooth : CA_Digit	al _CA_11	
	Tx02	08.05.2020 09:00:56			Light Diffuse:	220 🛞 💌	Tooth 2	_	
					Material Diffuse:	192	Tooth 2		
					Preventer Criticae.	190	🙎 Tooth :		
					Move Crown	Ŕ	Tooth :		
					Select Tooth	• • 🕅	Tooth :	_	
	Object Name						Tooth :		
10	Tx03								
	Approved				Strip Mesial	🕂 🖃 🗉 mm	Attachment Naviga	tor	-
	Send as Container				Strip Distal	+ - = mm			
		1/			± Inclination	+ - = •			
	8	Save 🔀 Cancel		1	± Angulation	+ •			
1					± Rotation				
					# Mesial	+ - = mm			
50 m				l	A Vestbular	+ - = mm			
				×	* Occlusal	+ - = mm	Height	Distance	11
	01	02 0	03 04	0 5	Gingiva Simulat	ion A	Crown Navigator		
0	01	02 , 0	, 04	,	Method		Additional Objects		
_	1	1	1			-	Occlusogram		
me	: 8\$, 8\$ ID: 21062020001 Date of Record: 18.0:	3.2020 Ape: 9.9 Protect: Tx03		Save specific	planning		As Ending 🔹	O O	ĸ
		Save specific planning situation as Finding		Create Anim					
		Shadaon as	i inang		how Attachments				

Option [Save Project]

- Any partial or complete planning process can be saved as a project using the [Save Project] icon button on top
- Any number of projects can be saved side by side
- Such project contains the entire planning process that has been carried out up to this point
- Re-opening a project will restore the identical situation
- All projects are saved with its "parent" finding (the one that is used to carry out planning)

Option [Save Finding]

- Any specific planning situation can be exported as "child" finding by button [As Finding] bottom right
- Any number of specific findings can be exported
- Such findings representing the specific case situation at this point will be displayed as separate thumbnail on tab |images|
- Optionally (if available) movement animation and attachments are included
- Such "child" findungs can be used as "parent" findings for other modules*



2024/02/01 20:13

🔤 Onyo	(Ceph ^{3ns} 3D Lab - B‡, B‡ - 21062020001									-	o x
a								License	Release	Support •	🕮 Language
	Practice Name kline europe	• 👸	Patient 21062020001: 8\$, 8\$		•	Pat	ient Images	Presentation L	etters Form	ns Online	Recycle Bin
•	🖪 🖳 💌 🎤 险 🖭	8 8 X 🛛	🖲 🕅 🗶 🚺	# 🗄 🗖		A A 🗉	2D D	ata 3D Data		ö • 🖂	1 • 0
-20	ź	40	é	80 mm		2D: 0 3D: 2 18.03 Cast Perm	Addar-	Observation-18. 18.03.2020 Cast Permanent			
-60 mm											
					<u>}_</u> ×						
		S •									
Name: 8	1, 61 ID: 21062020001 Brthday: 24.03.3	2010 Age: 9,9 Created:	08.05.2020 Changed: 00	1.05.2020		104	3 V Wo	rkstation: User [RK8]	Server: ONY	XD83-[DAVID-U	G]@DAVID-LG

Case Approval

See topic Case Approval

From: http://www.onyxwiki.net/ - [OnyxCeph³™ Wiki]

Permanent link: http://www.onyxwiki.net/doku.php?id=en:saveplanning

Last update: 2020/05/10 15:37



