

Settings




System Options

In the modal (stay-on-top) Window System Options the user can set up, edit and modify most of the standard settings essential for the local and global application environment. Different settings are thematically pooled on separate sections which can be accessed by vertical ident keys (tabs). Settings and parameters modified by the user need to be confirmed and will be saved in the database thereby. Local settings will be saved according to the used local Windows™ login and local computer name as separate configuration ([INI-Settings](#)) in the Onyx database on the server.

The Window can be opened by

- Left mouseclick main menu: Options|System Options
- Right mouseclick main menu: Systemoptionen
- Keyboard Shortcut [CTRL]+O

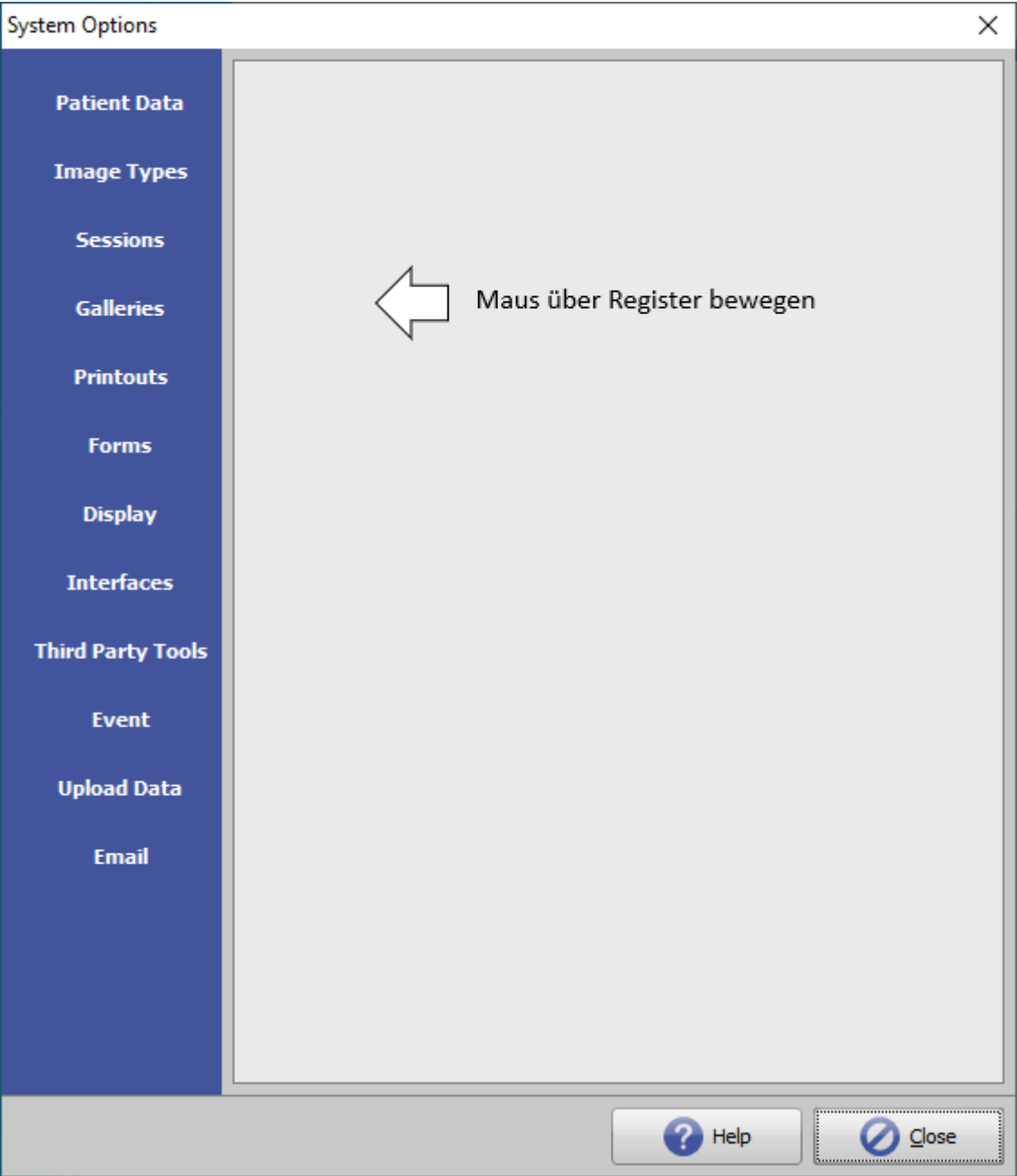

[Click on tab to switch to related topic]

	Tab	Settings / Konfiguration ...
1	Tab Patient Data	Patient-Thumbnails, VDDS settings, Markers, Misc ...
2	Tab Image Types	2D and 3D Image Type Settings
3	Tab Sessions	2D View Settigs 1/2/3, 3D View Settings
4	Tab Galleries	Galleries, Presentationen, Sequences, Media, View Option ...
5	Tab Printouts	Printform Templates, Letter Heads, Printer Preconfiguration ...
6	Tab Forms	DHTML-Forms by Categories, Password Forms in Fullscreen Mode ...
7	Tab Digitize	View Otions, User Analyses, Use Drawing Templates, Customization Landmark Sequence ...
8	Tab Interfaces	Configuration External Image Sources, External Documents Archive ...
9	Tab Third Party Tools	Configuration 3rd Party Tools
10	Tab Event	Event Settings definieren
11	Tab Upload Data	FTP-/WEB-Server Upload Settings for Containers, Reports, Webviews
12	Tab E-Mail	Email Client and Attachments konfigurieren

INI Settings



INI Settings should only be edited/modified by experienced professionals having sufficient knowledge about the anticipated result of the modification. Inappropriate changes to the INI, adding or deleting entries and/or parameters may result in malfunctioning or failure of the local clients application.

Local settings for Onyx client applications are saved separately for Windows™ login and computer name in a configuration record (Syntax: Login[Workstation]) in the Onyx database. With respect to former Windows™ versions such client configuration record is called "INI".

If the Onyx main context menu is opened by pressed [CTRL] button, context menu item [Configurationen] will be displayed to open the [Onyx Configurationen](#) window. All client configurations stored in the database can be displayed, edited, assigned and managed.

The used configuration can be opened and modified in an INI edit window also by button combination **[SHIFT]+[CTRL]** and **left mouseclick** on the help icon top right in the program main window.

INI Section	Setting for ...
Section [Application]	
Section [Aligner]	
Section [CASmart]	
Section [DigRoentgen]	
Section [Docmanager]	
Section [Export 3D]	
Section [FABonding]	
Section [ImageAcquire]	
Section [ImageTypeProps]	
Section [ImageTypes]	
Section [InfoServer]	
Section [Kommunikation]	
Sektion [Kylix]	
Section [Media]	
Section [MountCast]	
Section [PatientSelect]	
Section [Print]	
Section [PrintEditor]	
Section [Settings]	
Section [Surgery]	
Section [ThumbOC]	
Section [VirtualSetup]	
Section [Waefer]	

Skin Colors

After initial installation, the program will appear with a grey standard skin color. By activation, the skin color will switch to the standard blue user interface or - in case a customized branding is assigned with the activation (e.g. for program version OnyxCeph³™ CA-Smart 3D), another skin color can appear.

From:

<https://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:

<https://onyxwiki.net/doku.php?id=en:settings>

Last update: **2023/02/16 08:58**

