

Silent Client Setup

OnyxClientSetup supports silent mode using the following command line parameters:

Parameter	Value	Description
-?		Help desk for command line parameters
-Auto		Installation starts automatically
-Lang=	Language Number	For client language use: 1=DE, 2=EN, 3=RU, 4=ES, 5=IT, 6=FR, 7=CZ, 8=HU
-Server=	Server Name	z.B.: [MYSERVER]@MYSERVER oder [MYSERVER]@172.192.1.250
-Port=	IP-Port number	Optional; Default IP-Port number=16300
-PATH=	Install Folder	Local install folder or Default

- If any of the parameters is missing, OnyxClientSetup will stop at this point
- For **-PATH=Default** folder ..\Programs\Image Instruments\Onyx3Client\ is used
- If not specified, subfolder Onyx3Client will be created automatically
- In case of empty spaces in folder name, use quotation marks: **-PATH="C:\Onyx 3Ceph"**

Example

The command line below connects to the server via port 16300 and installs a client with GERMAN language settings in the standard installation folder:

```
OnyxClientSetup.exe -Auto -Server=[SRV]@SRV -Lang=1 -Path=Default
```

The command line below connects to the server via port 16500 and installs a client with ENGLISH language settings in folder D:\OnyxCeph\Onyx3Client:

```
OnyxClientSetup.exe -Auto -Server=[SRV]@172.1.1.250 -Port=16500 -Lang=2 -Path=D:\OnyxCeph
```

From:

<http://onyxwiki.net/> - **[OnyxCeph³™ Wiki]**

Permanent link:

<http://onyxwiki.net/doku.php?id=en:silentclientsetup>

Last update: **2023/10/24 10:01**

