Interface Configuration Digital Xray: VisiQuick

igital X-Ra	y						×
CliniVie	CliniView DBSW		Vin	VistaScan		Ro	mexis
VixWin	VixWir	n Scan	Sidexis	Orex	Dima	xis	Digora
Easy	Dent		Cares	tream		Scanora	
Orthometric/3DentStu		idio	VisiQuick		V	atech	
ñě	Vis	iQuick - S	Support /	Activated			
VisiQuick	- Progra	am Folde	er:				
C:\Te	mp\xra	y\VisiQu	iick.exe				
VisiQuick I	Databa	se					
-							
Show	Preview	w Image	s				
Eit to	VisiOuir	-k -					
	visiQuit	_K					

Parameter	Setting
[Support Activated]	Activates import icon button in module Add Image 2D
Program Folder []	Local folder for executable\VisiQuick.exe
Database []	Path for the VisiQuick data base
[Show Preview Images]	Activates thumbnail preview in preview panel module Add Image 2D
[Fit to VisiQuick]	Adjusting ID

From: https://onyxwiki.net/ - [OnyxCeph^{3™} Wiki]

Permanent link: https://onyxwiki.net/doku.php?id=en:visiquick



