


Window Image Attributes and Comments

The "Image Attributes and Comments" window can be opened using the Attributes / Comments icon button or via the thumbnail context menu [Image Attributes]. The window also appears when you save a finding with the [Save as finding] button..

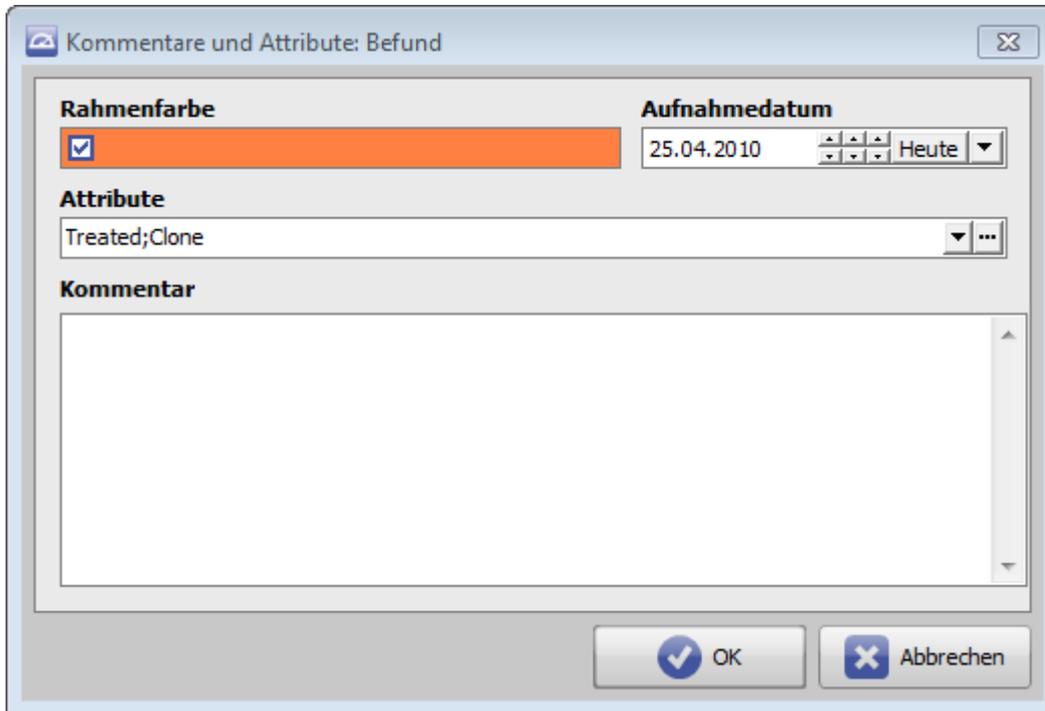

Thumbnail Frame Color

The selected color defines the inner thumbnail frame color.



Record Date

The record date is initially taken over from the imported image file but can be changed/adjusted if needed.

Image Attributes

Image attributes can be assigned from the attribute line pull down [▼].

Available		Assigned	
3D Object	3D Object	Treated	Treated
Aligner	Aligner	Adjust3D	Adjust3D
Bosboom	Bosboom	Segment	Segment
CAStep_0	CAStep_0	Completed	Completed
CAStepCMan2	CAStepCMan2	AutoTraced	AutoTraced
CAStepCMax3	CAStepCMax3	VTO 3D	VTO 3D
Ceph2D	Ceph2D		
Clone	Clone		
Combine	Combine		
Compare	Compare		
Edit	Edit		
FA-Bonding	FA-Bonding		

The list of available attributes can be modified via browse button [...]


Abbreviation	Description
3D Object	3D Object
Adjust3D	Adjust3D
Aligner	Aligner
AutoTraced	AutoTraced
Bosboom	Bosboom
CAStep_0	CAStep_0
CAStepCMan2	CAStepCMan2
CAStepCMax3	CAStepCMax3
Ceph2D	Ceph2D
Clone	Clone
Combine	Combine

Image Comments

In addition to pre-defined attributes, text comments can be added.

Display and Search

The presence of attributes and / or comments is indicated by a marker in the top right of the findings thail. If you mouse over a report thumbnail, the assigned attributes and / or comments are displayed in the back.

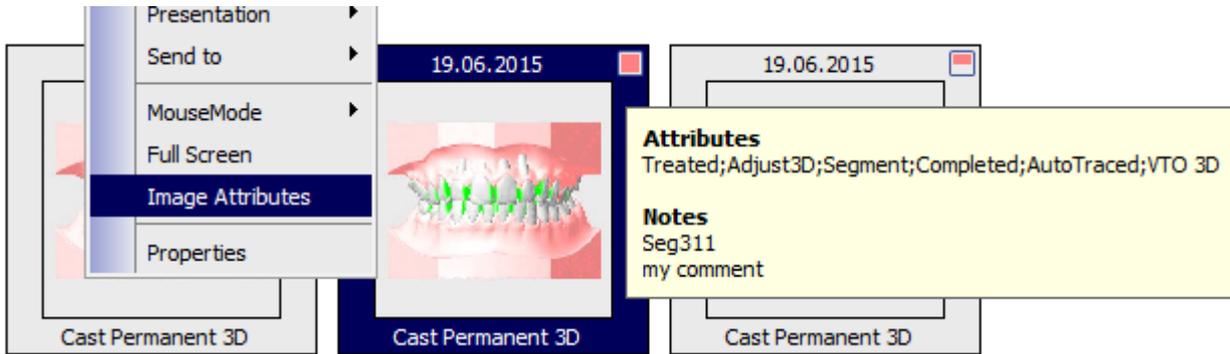

Attributes and comments can be identified in a full-text search by highlighting the relevant report thumbnails in blue. The search line is opened with [CTRL] + F. In addition, the [OnyxStat](#) sub-program offers the option of cross-patient filtering according to image attributes for statistical analysis purposes.



From: <https://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link: https://onyxwiki.net/doku.php?id=en:wdw_imageattributes

Last update: 2020/03/16 10:27

