

Befundvergleich 3D

OnyxCeph™ - Module Image Comparison

	18.10.2017	30.06.2018	18.10.2017	18.10.2017	30.06.2018 (3D)	19.10.2017 (3D)	30.06.2018 (3D)
A-Max	48,3mm	51,1mm	48,8mm	48,0mm	-3,1	0,8	-2,8
B-Max	49,3mm	50,3mm	49,3mm	49,3mm	-1,0	0,8	-0,8
D-Max	-1,2mm	-0,8mm	-1,2mm	-1,2mm	-2,0	0,8	-3,1
A-Min	47,4mm	50,5mm	47,4mm	47,4mm	-3,1	0,8	-3,0
B-Min	49,2mm	50,2mm	49,2mm	49,2mm	-1,0	0,8	-0,1
D-Min	-1,8mm	-0,3mm	-1,8mm	-1,8mm	-2,1	0,8	-2,9
A-Mean	44,3mm	44,4mm	44,3mm	44,3mm	-0,1	0,8	-1,1
B-Mean	49,5mm	49,2mm	49,5mm	49,5mm	-0,7	0,8	-0,2
D-Mean	-1,3mm	-1,6mm	-1,3mm	-1,3mm	0,8	0,8	-1,0
A-Min-Max	45,1mm	45,4mm	45,1mm	45,1mm	-0,3	0,8	-0,8
B-Min-Max	48,1mm	49,8mm	48,1mm	48,1mm	0,3	0,8	0,8
D-Min-Max	-0,9mm	-0,9mm	-0,9mm	-0,9mm	-0,4	0,8	-1,3
A-archMax (3-5)	48,3mm	48,9mm	48,3mm	48,0mm	-0,9	0,8	-0,9
B-archMax (3-5)	50,6mm	50,3mm	50,6mm	50,6mm	0,3	0,8	-0,2
Avd. Ratio (3-5)	82,8%	80,0%	82,8%	82,8%	2,3	0,8	1,1
B-archMax (5-6)	58,5mm	58,5mm	58,5mm	58,5mm	-2,0	0,8	-0,6
B-archMax (3-6)	51,6mm	52,1mm	51,6mm	51,6mm	-0,5	0,8	0,4
Ch. Ratio (3-6)	93,0%	91,6%	93,0%	93,0%	1,4	0,8	1,0
Ratio A-Max / B-Max	94,0%	98,0%	94,0%	94,0%	3,2	0,8	3,8
Ratio A-Max / D-Max	93,4%	98,0%	93,4%	93,4%	5,4	0,8	3,4
Ratio A-Max / B-Min	93,7%	98,4%	93,7%	93,7%	3,3	0,8	3,5
Ind. Method							
Symmetry	None (Standard)	None (Standard)	None (Standard)	None (Standard)	-	-	-
Center Dental Midpoint	Yes (Standard)	Yes (Standard)	Yes (Standard)	Yes (Standard)	-	-	-
Healthy Max. Ant. Teeth					-	-	-
Healthy Mand. Ant. Teeth					-	-	-

Load

Save

Help

Close

Analysis

Treatment: 3D: Cast 3D Permanent

Analyst: Onyx Cast 3D

Add Columns

Findings

All Findings

All, With Clinical Norms

Buttons

Delete Column

Columns Colors

Acc. to Sequence

Acc. to Finding State

Progress

From:
<https://www.onyxwiki.net/> - [OnyxCeph3™ Wiki]

Permanent link:
https://www.onyxwiki.net/doku.php?id=image_comparison_3d&rev=1447426156

Last update: 2015/11/13 15:49

